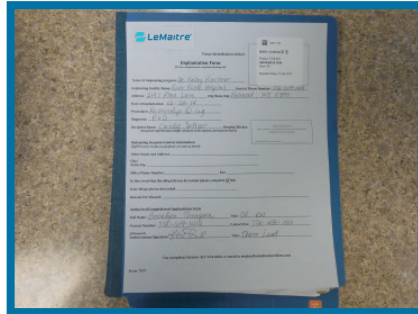


TISSUE IMPLANTATION FORM

- 1). At the completion of the tissue implantation the Tissue Implantation Form shall be completed and returned back to LeMaitre Vascular, Inc. This form is found inside the Tissue Implantation Packet.
- 2). Form shall be sent back to LeMaitre Vascular, Inc. via FAX at 847-277-7273 or go to www.lemaitre.com/implantform. Completed forms can also be sent back with cryogenic shipper.

RETURN OF EMPTY CRYOGENIC SHIPPER

- 1). Remove courier airbill located in the back of the Tissue Implantation Packet, attach to the external shipper top.
- 2). Replace all items provided with shipper (i.e. cryogloves) back into shipper and secure.
- 3). Contact courier as noted on airbill for pickup of shipper.

**RETURN OF CRYOGENIC SHIPPER WITH TISSUE**

- 1). If tissue is to be returned back to LeMaitre Vascular, the seal tag must be intact and not broken.
- 2). Remove courier airbill located in the back of the Tissue Implantation Packet and attach to the top of the shipper.
- 3). Replace Tissue Implantation Packet and all items provided with shipper (i.e. cryogloves) back into shipper and secure latch.
- 4). Contact courier as noted on airbill for pickup of shipper.

Note: If the seal tag has been broken and tissue was not used, a LeMaitre Vascular Representative **MUST** be notified to approve shipment.

