


# COMPLETING TISSUE IMPLANTATION FORM AND RETURNING CRYOGENIC SHIPPER


## Tissue Implantation Form

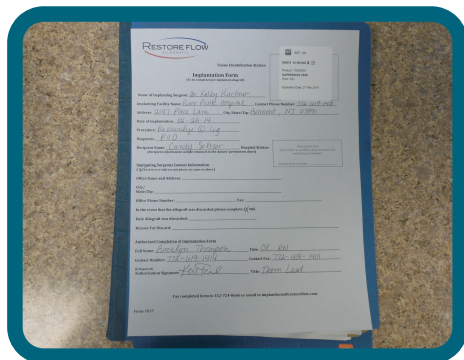

- 1) At the completion of the tissue implantation the Tissue Implantation Form shall be completed and returned back to LeMaitre Vascular, Inc. This form is found inside the Tissue Implantation Packet.
- 2) Form shall be sent back to LeMaitre Vascular, Inc. via FAX at 847-277-7273 or go to [www.lemaitre.com/implantform](http://www.lemaitre.com/implantform). Completed forms can also be sent back with cryogenic shipper.

## Return of Empty Cryogenic Shipper

- 1) Remove courier airbill located in the back of the Tissue Implantation Packet, attach to the top of the mushroom shipper.
- 2) Replace all items provided with shipper (i.e. cryogloves) back into shipper and secure latch.
- 3) Contact courier as noted on airbill for pickup of shipper.

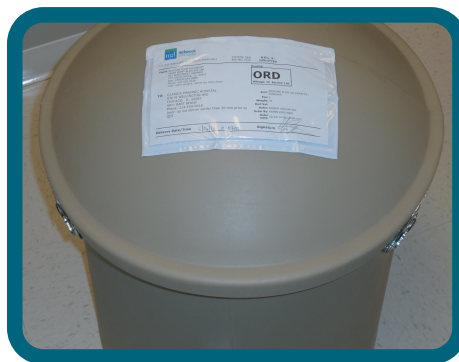

## Return of Cryogenic Shipper with Tissue

- 1) If tissue is to be returned back to Restore Flow Allografts, the seal tag must be intact and not broken.
- 2) Remove courier airbill located in the back of the Tissue Implantation Packet and attach to the top of the mushroom shipper.
- 3) Replace Tissue Implantation Packet and all items provided with shipper (i.e. cryo- gloves) back into shipper and secure latch.
- 4) Contact courier as noted on airbill for pickup of shipper.

**Note:** If seal tag has been broken and tissue was not used, an RFA representative **MUST** be notified to approve shipment.

