**EndoHelix™ Retrieval Device**

Instructions for Use - English

**EndoHelix™ Retrieval Device**

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**EndoHelix™ Retrieval Device**

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**EndoHelix™ Retrieval Device**

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**EndoHelix™ Retrieval Device**

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**EndoHelix™ Retrieval Device**

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This procedure must be performed by personnel well trained in anatomical landmarks, safe techniques and potential complications. Perioperative fluoroscopic guidance is necessary to perform this procedure.

Do not alter the device during insertion, use or removal. Product is sterile only in unopened undamaged package. Inspect the package before opening. If package is opened or damaged, or if seal is broken, do not use.

**CAFULLY READ ALL OF THE INSTRUCTIONS PRIOR TO USE. OBSERVE ALL WARNINGS AND PRECAUTIONS NOTED THROUGHOUT THE INSTRUCTIONS. FAILURE TO DO SO MAY RESULT IN COMPLICATIONS.**

## Device Description

The EndoHelix™ Retrieval Device is designed to remove fractured core segments from the lumen of the artery with a variable diameter in order to better fit varying size arteries and atheromas. The EndoHelix Retrieval Device endarterectomy device is a tubular device with an adjustable helical wire at the distal tip. The device is manufactured with a stainless steel shaft, a polymer handle, a shape memory helical wire (NiTi), and stainless steel spatula tip to be used without a guidewire (reference Figure 1). The helical wire is adjustable to fit the size of the vessel and the atheroma. The working length of the device is 43 centimeters. The diameter of the shaft is 6 French. The maximum expanded diameter of the helical wire is 13mm.

## Intended Use/Indications

The EndoHelix Retrieval Device endarterectomy device is intended to be used to remove plaque deposits from arteries.

## Contraindications

- Calcification of Artery
- Absence of Patent Distal Artery
- Vessel size less than 6mm and greater than 11 mm
- Renal insufficiency preventing use of contrast media
- IV Contrast Allergy

## Warnings/Precautions

The EndoHelix Retrieval Device endarterectomy device is not for use in coronary or neuro vasculature. Do not use excessive force with this procedure. As with any other procedure performed without direct visualization of the vessel, extreme care and discretion must be exercised at all times. Complications associated with PTA and the use of endarterectomy devices include:

1. Vessel damage that may require replacement/ conversion to bypass
2. Vessel perforation and bleeding (hematoma, pseudo-aneurysm, injury, rupture)
3. Acute vessel occlusion due to spasm, thrombosis, or distal dissection
4. Limb ischemia or loss due to distal embolism or 3 above
5. Incisional/ puncture site inflammation or infection
6. Fever, drug reaction or allergic reactions to contrast medium, renal failure
7. Death

## How Supplied

**STERILE:** Each EndoHelix Retrieval Device endarterectomy device is sterile and individually packaged. The EndoHelix Retrieval Device endarterectomy device is sterilized using ethylene oxide (EO), and is for single use only. Do not attempt to reuse or resterilize. Do not use if package is opened or damaged.

**STORAGE:** Store in a cool, dry place. Proper stock rotation should be practiced.

## Clinical Use Information

### Inspection Prior To Use

Carefully inspect the sterile package for damage or defects before opening. Do not use the product after the "Use By" date on the package. If the integrity of the sterile package has been compromised or the product or the package is defective, do not use the product and contact your LeMaitre Vascular representative.

### Other Ancillary Devices For Procedure

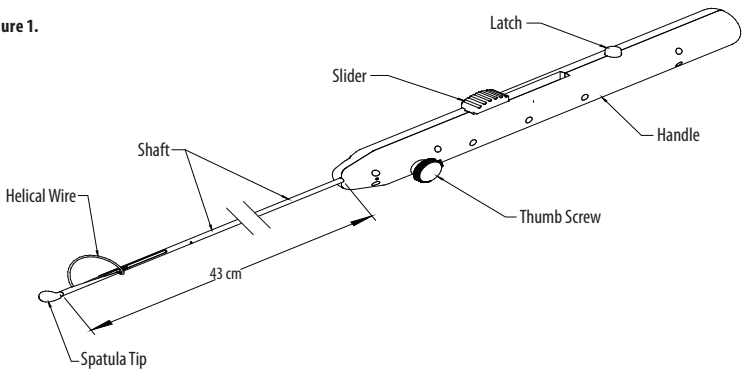
LeMaitre Mollring Cutter® Transection Device or comparable product.

## Device Operating Instructions

### Patient Preparation

Premedicate patient with appropriate analgesic and antibiotic per physician discretion. The patient should be prepped according to your institution's practice for vascular surgical procedures.

**Figure 1.**

**Directions For Use**

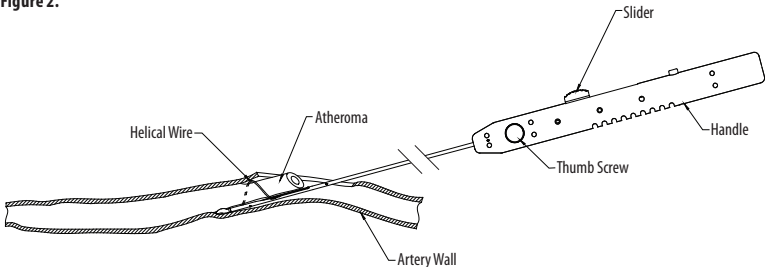
The exact technique for endarterectomy is determined by the physician. One suggested technique is as follows:

1. Once a standard endarterectomy (i.e. the dissection of core with either a dissection ring or a Martin Dissector, and the transection of distal core with a MollRing Cutter) has been completed, the EndoHelix Retrieval Device can be used to snare remaining length of fractured core segments in the artery.
2. An angiogram with contrast should be completed prior to using EndoHelix Retrieval Device to ascertain location and length of remaining core within the artery.
3. After location of proximal core has been established, remove EndoHelix Retrieval Device from packaging and remove the silicone sleeve from tip.
4. Pull snare wire flush and taut by pulling back on slider until it clicks into place. With wire pulled taut, the instrument is now ready for insertion into the artery through the existing arteriotomy.
5. On angiography, proceed with EndoHelix Retrieval Device down the artery until it is positioned approximately 2-3cm above the proximal end of the core.
6. Deploy snare wire at this time by pushing the latch on the handle. This will open the wire, and thereby prepare the instrument to snare the remaining core. See Figure 2.
7. Proceed further down the artery with the EndoHelix Retrieval Device, running the open wire around the circumference of the core. Continue until 3-4cm of core is within the wire on angiography. At this time, pull back again on the slider until it clicks into place, or until resistance is felt. The core is now snared and ready to be removed.
8. Pull the EndoHelix Retrieval Device, along with snared core, out of the artery through the arteriotomy. Repeat steps 1-8 until all core segments have been removed and artery is clean per angiography.
9. Flush the vessel with saline to clear any loose, residual material. Angiography, angioscopy or intravascular ultrasound may be used to verify vessel patency following the procedure.
10. Secure any flaps that may possibly dissect when flow is re-established.
11. Close the arteriotomy.

Note: Physician experience and discretion will determine the appropriate drug regimen for each patient.

Note: Do not excessive force as arterial damage may occur.

**Figure 2.**

## **Resterilization/Re-use**

This device is single-use only. Do not reuse, reprocess, or re-sterilize. The cleanliness and sterility of the re-processed device cannot be assured. Reuse of the device may lead to cross contamination, infection, or patient death. The performance characteristics of the device may be compromised due to reprocessing or re-sterilization since the device was only designed and tested for single use. The shelf life of the device is based on single use only.

## **Safe Handling and Disposal**

This device is single-use and disposable device. Do not implant. Please return the used device only at the time that the device has not performed as intended or the device is related to an adverse event. In other situations, the device should not be returned but disposed according to local regulations.

If serious medical incidents should arise during use of this medical device, users should notify both LeMaitre Vascular and the Competent Authority of the country where the user is located.

This product contains no sharps, heavy metals or radioisotopes, and is not infectious or pathogenic. No special requirements for disposal are evident. Please consult local regulations to verify proper disposal.

### **Cleaning:**

1. Devices considered necessary to return should be cleaned using one of the following:
  - a) Sodium hypochlorite solution (500-600 mg/l), or
  - b) Peracetic acid solution with subsequent ultrasonic treatment
2. Devices should then be decontaminated with either:
  - a) 70% solutions of ethanol or isopropanol for a minimum of 3 hours or,
  - b) Ethylene oxide gas
3. Devices should be completely dried prior to packaging.

### **Packaging:**

1. Cleaned devices should be sealed and packed in a manner that minimizes potential for breakage, contamination of the environment or exposure to those handling such packages during transit. For devices capable of penetrating or cutting skin or packaging material, the primary packaging must be capable of maintaining the product without puncture of the packaging under normal conditions of transport.
2. The sealed primary container should be placed inside watertight secondary packaging. The secondary packaging should be labelled with an itemized list of the contents of the primary receptacle. Cleaning methods should be detailed if possible.
3. Both primary and secondary packaging of cleaned, decontaminated single-use disposable devices should be labelled with an ISO 7000-0659 Biohazard symbol.
4. Primary and secondary packaging must then be packaged inside an outer package, which must be a rigid, fiberboard box. The outer shipping container must be provided with sufficient cushioning material to prevent movement between the secondary and outer containers.
5. Shipping paper and content marking for the outer shipping container are not required.
6. Packages prepared in the above manner may be shipped to:

LeMaitre Vascular  
Attn: Complaint Lab  
63 Second Avenue  
Burlington, MA 01803

## **Limited Product Warranty; Limitation of Remedies**

LeMaitre Vascular, Inc. warrants that reasonable care has been used in the manufacture of this device. Except as explicitly provided herein, LEMAITRE VASCULAR (AS USED IN THIS SECTION, SUCH TERM INCLUDES LEMAITRE VASCULAR, INC., ITS AFFILIATES, AND THEIR RESPECTIVE EMPLOYEES, OFFICERS, DIRECTORS, MANAGERS, AND AGENTS) MAKES NO EXPRESS OR IMPLIED WARRANTIES WITH RESPECT TO THIS DEVICE, WHETHER ARISING BY OPERATION OF LAW OR OTHERWISE (INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE) AND HEREBY DISCLAIMS THE SAME. LeMaitre Vascular makes no representation regarding the suitability for any particular treatment in which this device is used, which determination is the sole responsibility of the purchaser. This limited warranty does not apply to the extent of any abuse or misuse of, or failure to properly store, this device by the purchaser or any third party. The sole remedy for a breach of this limited warranty shall be replacement of, or refund of the purchase price for, this device (at LeMaitre Vascular's sole option) following the purchaser's return of the device to LeMaitre Vascular. This warranty shall terminate on the expiration date for this device.

IN NO EVENT SHALL LEMAITRE VASCULAR BE LIABLE FOR ANY DIRECT, INDIRECT, CONSEQUENTIAL, SPECIAL, PUNITIVE, OR EXEMPLARY DAMAGES. IN NO EVENT WILL THE AGGREGATE LIABILITY OF LEMAITRE VASCULAR WITH RESPECT TO THIS DEVICE, HOWEVER ARISING, UNDER ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, TORT, STRICT LIABILITY, OR OTHERWISE, EXCEED ONE THOUSAND DOLLARS (US\$1,000), REGARDLESS OF WHETHER LEMAITRE VASCULAR HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH LOSS, AND NOTWITHSTANDING THE FAILURE OF THE ESSENTIAL PURPOSE OF ANY REMEDY. THESE LIMITATIONS APPLY TO ANY THIRD-PARTY CLAIMS.

These limitations do not apply to consumers in Australia or to the extent they are precluded by local law in any other jurisdiction.

A revision or issue date for these instructions is included on the back page of these Instructions for Use for the user's information. If twenty-four (24) months has elapsed between this date and product use, the user should contact LeMaitre Vascular to see if additional product information is available.

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LeMaitre and MollRing Cutter are registered trademarks, and EndoHelix and Martin are trademarks, of LeMaitre Vascular, Inc.
©2019 LeMaitre Vascular, Inc. All rights reserved. Printed in the USA.
LeMaitre Vascular GmbH