VascuCel® Vascular Patch Instructions for Use

DESCRIPTION

VascuCel® is prepared from Bovine Pericardium.

The pericardium is procured from cattle originating in Australia. Australia is considered as being of negligible risk for BSE.

The device is a sterile, off-white, moist, pre-cut, flat sheet of acellular collagen, presented sterile in a solution of propylene glycol and sealed in a jar impermeable to air and moisture.

SIZES

The VascuCel vascular patch comes in various sizes.

STERILISATION

The VascuCel vascular patch is sterilized by exposure to, and storage in, a solution of 4% propylene oxide in sterile water (Osmolarity of ≤ 10 m Osm/L) which converts to propylene glycol solution.

INDICATIONS

VascuCel is indicated as a patch in great vessel repair, peripheral vascular reconstruction and suture line buttressing.

CONTRA-INDICATIONS

VascuCel is not designed, sold or intended for use except as indicated.

The use of a tissue bioprosthesis must be considered on an individual patient basis.

INTENDED USER

The intended users of this device are qualified Cardiothoracic, Vascular and General Surgeons.

INTENDED PATIENT POPULATION

The VascuCel device is designed for permanent implantation in humans, and is indicated for repairing or reconstructing cardiac and vascular deformities.

WARNINGS

To avoid damage to the product, do not expose to any substances other than those specified in the rinse procedure.

Do not freeze.

Do not expose the device to extreme heat. Do not re-sterilise.

Single use only and cannot be re-used.

Use of the device following a compromise in sterility may result in infection.

Do not use beyond the indicated expiration date.

Studies have not been performed to evaluate the safety or compatibility of this device during magnetic resonance imaging (MRI) scans. As such, the potential hazards of MRI testing on patients receiving this bioprosthesis are unknown.

PRECAUTIONS

Store the package the right-side up.

The outside of the jar is not sterile and must not be placed in the sterile field.

Do not use the device if the tamper-evident seal is broken.

Do not use the device if the Freeze indicator inside the lid of the device package indicates that the device has been tripped.

Do not use the device if there is evidence of damage to, or leakage from, the jar, or if the solution appears turbid as sterility of the product may have been compromised.

Do not expose the patch to any solutions, chemicals, antibiotics, antimycotics, or other drugs except for the storage solution or sterile physiological saline, as irreparable damage to the patch may result that is not apparent under visual inspection.

Rinse surgical gloves to remove glove powder prior to touching VascuCel.

Prior to surgery, prospective patients or their representatives should be informed about possible complications which may be associated with the use of this device.

As with any surgical procedures, infection is a possible complication. Monitor patient for infection and take appropriate therapeutic action.

PREPARATION FOR SURGICAL USE

Do not use if jar has evidence of having previously been opened or damaged.

First Handling

Remove inner jar from outer cardboard package and polystyrene inserts. Do not place the jar in the sterile field. Inspect jar. Do not use if the tamper proof seal is missing or damaged, or there is evidence of moisture or leakage from the jar. The contents must be handled using aseptic techniques.

Opening

Open the jar without contaminating the contents. A member of the sterile team should remove the patch using sterile atraumatic forceps to grasp the edge of the patch and remove from the jar. Immerse in sterile physiologic saline.

Hydration

No extensive rinsing is required. Immersing the patch in sterile physiological saline is required to avoid dehydration of the patch prior to implantation.

THE DEVICE MUST REMAIN MOIST AT ALL TIMES.

Presentation to Surgeon and Use

The implant should be presented to the surgeon using sterile atraumatic forceps.

Surgical Use

The surgeon may cut and shape VascuCel to suit the requirements of the procedure.

Care should be taken when handling the device, for example by using atraumatic forceps, to avoid tearing or otherwise damaging the patch.

The device should be visually examined for damage noting it may have a smooth and a rough side.

VascuCel may be cut, folded or layered as required. If layering, it is preferable to cut the material into separate sheets, creating edges rather than to fold it, presenting the maximum number of cut surfaces to body tissue, to enhance penetration by cells and blood vessels.

VascuCel may be sutured or stapled in place. Being a strong material, it will take and hold sutures easily and rmlly, and will remain in situ while it is incorporated into surrounding tissue.

When implanting by suture, suture bites should be taken 2 to 3 millimetres from the edge of the patch.

Note: After removal from its jar, all unused pieces of VascuCel should be discarded.

The solution in which the VascuCel is stored can be disposed of according to hospital procedures for non-hazardous materials.

STORAGE

VascuCel should be stored at room temperature, never below 2°C or above 25°C, and away from direct heat source. The implant should not be re-sterilised.

POTENTIAL COMPLICATIONS

Restenosis, pseudo-aneurysm formation, infection, thrombosis, calcification, fibrosis, vessel occlusion, patch rupture, dilatation, myocardial infarction, bleeding, stroke, death.

ADVERSE REACTIONS

The effect of long term surgical outcome of damaging bovine pericardium by chemicals or substances other than saline, by freezing, or by exposure to extreme heat, or chemical sterilisation has not been investigated.

When bovine pericardium has been used for pericardial closure, cases of epicardial inflammatory reactions have been reported.

Other adverse events associated with bioprosthetic pericardial patches that have been reported

in the literature have included: calcification; haemolysis; flow obstruction; thromboembolism; endocarditis; pericardial adhesions; inflammation; degeneration of the implants; and formation of clinically significant fibrous tissue.

PATIENT REGISTRATION

Anteris is dedicated to monitoring the post market performance of VascuCel implants. To assist in achieving this goal, we include an Implantation Card; please complete and return it to Anteris Aus Operations Pty Ltd. A permanent record of all VascuCel implantations can then be maintained.

DISCLAIMER OF WARRANTIES

Anteris Aus Operations Pty Ltd warrants that reasonable care has been used in the manufacture of this device. This warranty is exclusive and in lieu of all other warranties whether expressed, implied, written or oral, including, but not limited to, any implied warranties of merchantability or fitness. Due to the inherent biological differences in individuals, no product is 100% effective under all circumstances. Bioprosthesis failure may occur for a variety of reasons, both medical and physical, including inadvertent damage to the device before, during and/or after clinical implantation.

As Anteris Aus Operations Pty Ltd has no control over the conditions in which the device is used, diagnosis of the patient, methods of use, or its handling after the device leaves its possession, Anteris Aus Operations Pty Ltd does not warrant either a good effect or against ill effect following its use, nor will it be liable for any incidental or consequential loss, damage, or expense arising directly or indirectly from the use of this device.

As some jurisdictions do not allow the exclusion of, or limitations in, an implied warranty, or do not allow the exclusion or limitation of incidental or consequential damages, the above exclusions may not apply.

SYMBOL DEFINITIONS			
	Contents are Sterile		Sterilized using Propylene Oxide
	Single use only. Do not re-use.		Product derived from Australian cattle.
	Attention! See Instructions for Use.		Thickness
	Minimum and maximum storage temperatures		Consult instructions for use.
	Use by (with date)		Do not resterilize.
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.		Do not use package if product is damaged.
	Catalogue Number		Date of manufacture
	LOT Number		Manufacturer
	Serial Number.		

VascuCel and LeMaitre are registered trademarks of LeMaitre Vascular, Inc.

©2023 LeMaitre Vascular, Inc. All rights reserved.

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